



2. HOW TO RUN MASTATEST

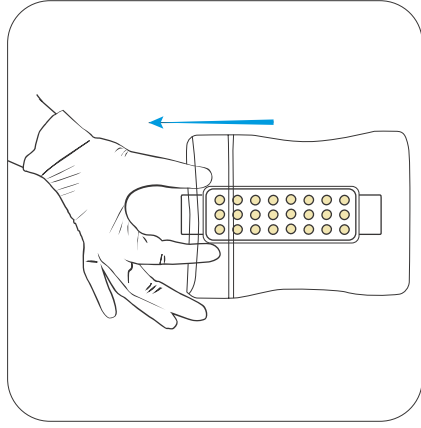


1. OPEN

Wearing clean gloves, remove cartridge from pouch.

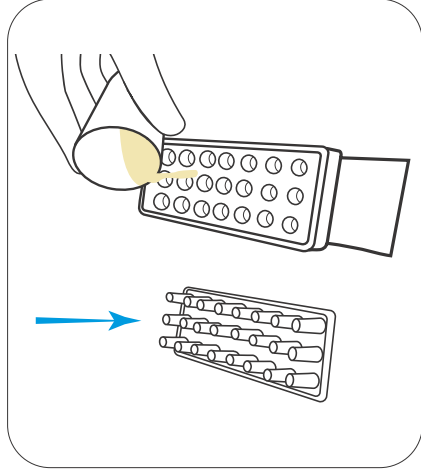


2. ADD MILK SAMPLE

Remove lid and place lid upside down (ensure you do not touch the plungers). Fill the Mastatest® with milk until half full.

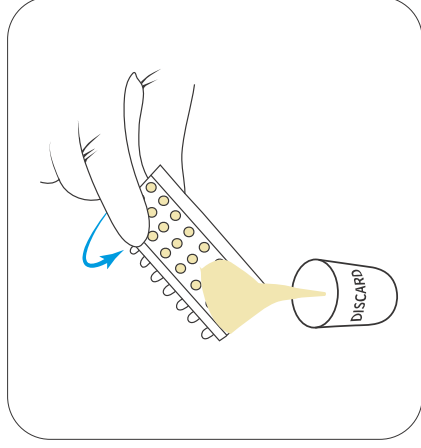


3. DISCARD SURPLUS MILK

Pour off surplus milk, and discard. Do not put surplus back into milk sample.

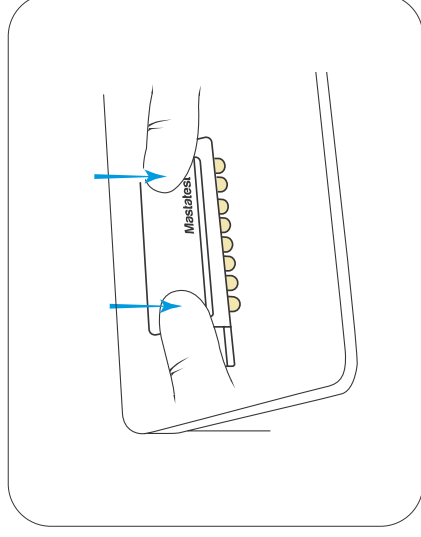


4. REPLACE LID

Replace lid (ensuring it is centered) and push down firmly onto bench.

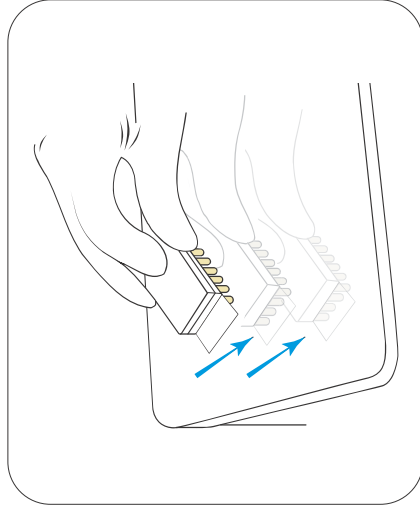


5. TAP FIRMLY

It is important to tap firmly, twice.

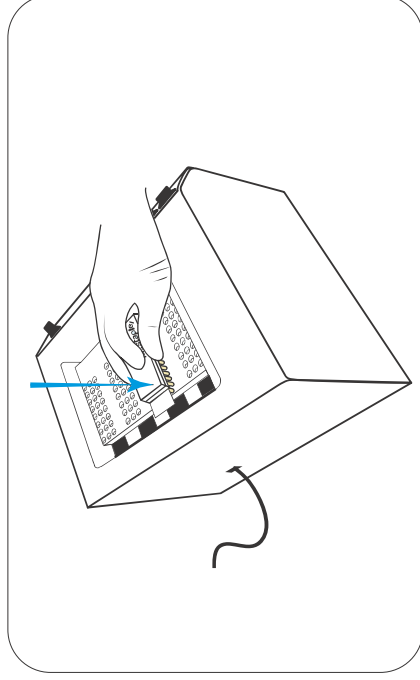

6. INSERT CARTRIDGE

Insert into Mastatest® lapbox with QR code label facing down and toward the back of the lapbox.

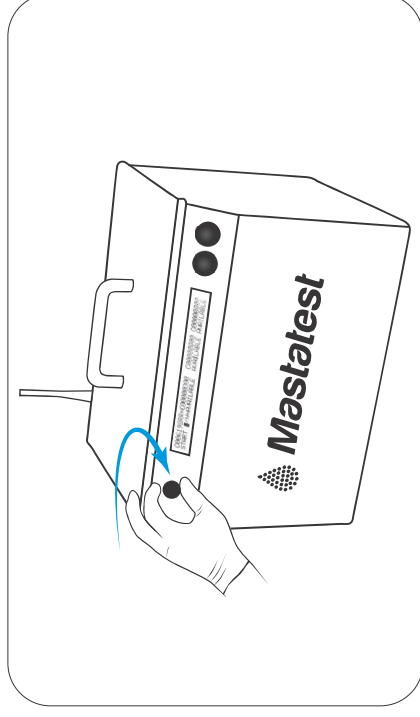

7. START

Dial in the cow's number. Press enter to start test. Results sent via email within 24 hours.

- It is important not to contaminate the milk sample
- Wear clean gloves
- Keep all work surfaces clean by wiping with teat wipes
- Keep out of reach of children
- Veterinary product for animal use only
- After use drop the test in bleach before disposing in a suitable container