

Set-up Guide



1. Connect

- Connect the power cable, and ethernet cable (if available)

2. Set preferences

- Set operating language: scroll ▼, then ENTER to confirm
- If no ethernet is detected:
 - ENTER to scan for Wifi networks, scroll ▼ to select, then ENTER to confirm
 - Input password: Use number keys, or scroll ▼ ▲ to select characters. Move cursor using ◀ ▶
 - ENTER to save password, then to continue ▲ ENTER
- Note 16-digit Lapbox ID, then ENTER to exit set-up

3. Register Lapbox

For new clients: Create your account at mastatestsystem.com

For existing customers:

- Login to your mastatestsystem.com account
- Select "My Account" and "Register a Lapbox" from menu

Caring for your Mastatest system

- Recommended room temperature of 5-30°C (41-86°F) for best performance
- Position Lapbox on a clean, dry desk space and handle with care
- Keep internal and external surfaces clean using a damp cloth (water only)
- Store cartridges at 2-8°C (35-46°F), and use before expiry date



For information in your language vetoquinol-mastatest.com/how-to-use
Deutsch, Français, English, Español, Italiano, Portuguese, Polski, Nederlands

User Information

- Red light indicates power on.
- Green light indicates internet connection.
- to return to previous screen.
- long-press to delete a character entry.

Set-up menu options are always available by pressing when on main testing screen