

## Fill a Mastatest® cartridge




Take a clean milk sample  
(see user videos)




Put on clean gloves, and remove  
cartridge from pack



Remove cartridge lid and  
half-fill cartridge with milk

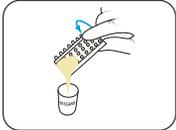

Pour off surplus milk and discard

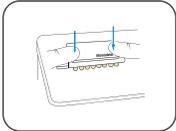

Replace lid, pushing down firmly,  
and label lid with cow details



Tap firmly 2-3 times, repeating if  
air-bubbles are visible in wells



Insert cartridge into Lapbox™  
with arrows facing to the back  
and replace lid

## Start a Mastatest®

1. Start position defaults to first available. **ENTER** to confirm,  
or scroll **◀ ▶** to chosen position, then **ENTER**

*Cartridge type should be automatically detected.*

*If not select Single (1 test) **ENTER**  
or Twin (2 tests) cartridge **▼ ENTER***

2. Select **BoxStart **ENTER****, or **WebStart **▼ ENTER****

3. Enter animal details:

- BoxStart: Enter animal number eg: **1 2 3 **ENTER****  
and scroll to select quarter **▼ **ENTER****
- WebStart: Enter at [mastatestsystem.com](http://mastatestsystem.com)

- When test has started, a countdown timer shows  
on screen. Further tests can be started at any time.
- **↶** to return to prior screen.
- Long-press **↶** to delete.
- To check or cancel a test in progress, scroll to select  
position **◀ ▶ **ENTER****, then **ENTER** to check or **▼ **ENTER**** to cancel.

